

Transparent Island Dressing




Instructions For Use:

- Select the appropriate size of the dressing.
- After clean and disinfect the wound, take out of dressing from the bag.
- Peel off the back release paper, exposing the adhesive surface.
- The adhesive surface paste to the skin surface, smoothing from the middle to the surrounding.
- Peel off surrounding release paper.

Features:

- Material: PU and cotton.
- With paper frame.
- Size: 6x7cm, 7x10cm, 10x10cm, 10x12cm, 10x15cm, 10x20cm, 10x25cm, 15x15cm.
- Cotton pad: absorb exudates from wounds.
- Wound will be not bonded to dressing.
- Thin and high transparent, clear observation to surgical operation part.
- Good permeability, air permeable, block bacteria, protect the wound from bacteria intrusion.
- Apply in intravenous injection in hospital, catheter fixing, and treatment and protection of the wound.

Size	Ref	Material
6cmx7cm	184-1-16A	PU and cotton
7cmx10cm	184-2-16A	PU and cotton
10cmx10cm	184-3-16A	PU and cotton
10cmx12cm	184-4-16A	PU and cotton
10cmx15cm	184-5-16A	PU and cotton
10cmx20cm	184-6-16A	PU and cotton
10cmx25cm	184-7-16A	PU and cotton
15cmx15cm	184-8-16A	PU and cotton

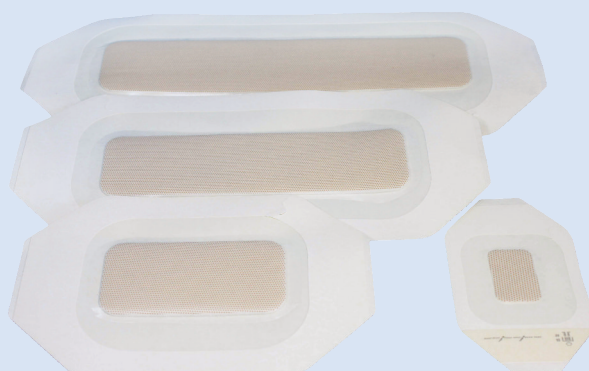

Caution:

2. This product is sterilized for single use only, discard after single use.

Please do not repeat sterilization, do not use it once again.

2. Should be used the product immediately after opening the individual package, please destroy when it was used.

3. Do not use it if the inner package has been previously opened or damaged, or package be affected with damp, or the product went moldy.

4. Do not use after expiry date. If over the expiry date, please discard it.

5. Sterile by ETO. Single use only. Latex free.

Storage:

Store in a cool, dry and dark place. Avoid direct sunlight.

Transparent Island Dressing



Secured Medical Direction UK Co., Ltd.

Wayson Commercial Bldg 28 Con-naught Rd West Sheung Wan HK, P.R.C.



Secured Medical Direction UK Co., Ltd.

St. James House 13 Kensington Square W8 5HD London U.K.

www.smd-medical.com