

Triangle Bandage



Instructions For Use:

- Do first aid and fasten the wrapping of wound area.
- Unfold the triangular bandage into correct shape.
- Hold the hurt arm by the bandage, and fasten behind the neck.
- It is multi-purpose use as:

Immobilizing broken or injured limbs.

As a wound dressing when no other bandages are available.

As a general purpose fabric dressing.

It can be used for military or civilian purposes.

It is an integral part of every first aid kit at home or at work. Always be prepared for emergency.

Features:

- Material: 100% cotton, or non-woven.
- Size: 36" x 36" x 51", 40" x 40" x 56".
- Air permeability, comfortable.
- Different length can be as required.

Size	Ref	Material
36" x 36" x 51"	233-1-16B	Nonwoven
40" x 40" x 56"	233-2-16B	Nonwoven
36" x 36" x 51"	234-1-16B	Cotton
40" x 40" x 56"	234-2-16B	Cotton



Caution:

1. This product is for single use only, discard after single use.
2. Do not use it if the package be affected with damp, or the product went moldy.
3. Do not use after expiry date. If over the expiry date, please discard it.
4. Single use only. Latex free.
5. Sterile or non-sterile, it can be as request.

Storage:

Store in a cool, dry and dark place. Avoid direct sunlight.

Triangle Bandage



Secured Medical Direction UK Co., Ltd.

Wayson Commercial Bldg 28 Con-naught Rd West Sheung Wan HK, P.R.C.



Secured Medical Direction UK Co., Ltd.

St. James House 13 Kensington Square W8 5HD London U.K.

www.smd-medical.com