


# Wooden Tongue Depressor

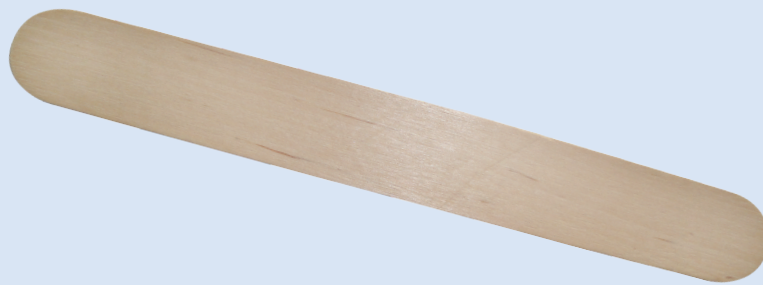

## Instructions For Use:

- Peel off the bag, take out the wooden tongue depressor.
- Press the front 2/3 of the patient's tongue.
- With the help of the light.
- Exam the tonsil, palatoglossal arch, pharyngopalatine arch, posterior wall etc.

## Features:

- Material: white birch wood.
- Size: 15cmx1.8cmx1.6mm, 14cmx1.4cmx1.6mm.
- Sterile or non-sterile.

Type	Size	Ref	Material
Sterile	15cm x 1.8cm x 1.6mm	123-S-1-20A	white birch wood
Sterile	14cm x 1.4cm x 1.6mm	123-S-2-20A	white birch wood
Non-sterile	15cm x 1.8cm x 1.6mm	123-NS-1-20A	white birch wood
Non-sterile	14cm x 1.4cm x 1.6mm	123-NS-2-20A	white birch wood



### Caution:

1. Read instruction for use before using the device.
2. This product is for single use only, discard after single use. Please do not clean, do not use it once again.
3. Do not use it if the package has been affected with damp, or the product went moldy.
4. Do not use after expiry date. If over the expiry date, please discard it.
5. Single use only. Latex free.

### Storage:

Store in a cool, dry and dark place. Avoid direct sunlight.

# Wooden Tongue Depressor



**Secured Medical Direction UK Co., Ltd.**

Wayson Commercial Bldg 28 Con-naught Rd West Sheung Wan HK, P.R.C.



Secured Medical Direction UK Co., Ltd.

St. James House 13 Kensington Square W8 5HD London U.K.

[www.smd-medical.com](http://www.smd-medical.com)