

Hypodermic Syringe (without needle)



Instructions For Use:


- Used together with hypodermic needle. Peel off the sealed package, fix the needle on the syringe tightly.
- Detach the needle protective cap. Don't touch the cannula during using.
- Pressing and depressing the plunger in the syringe cylinder vertically axis till the syringe piston becomes soften, it is performed prior to use the product.
- Suck in the medicine solution into the syringe cylinder by pulling the plunger vertical axis in the syringe cylinder as needed.
- Take out the air bubble from the syringe cylinder by pushing the plunger forwardly vertically axis in the syringe cylinder.
- Pick with a needle and inject the medicine solution into the place according to the needs.
- Discard after the performance of medical doing.

Features:

- Luer lock or Luer slip.
- Size: 1ml, 2ml, 2.5ml, 3ml, 5ml, 10ml, 20ml, 30ml, 50ml, 60ml.
- 3 parts (2 part can be as required).
- Without needle.
- Latex or latex free.

Name of parts for Hypodermic Syringe (without needle)

Hypodermic Syringe		
Item No.	Name	Material
1	Plunger	PE
2	Barrel	PP
3	Piston	Latex or latex free

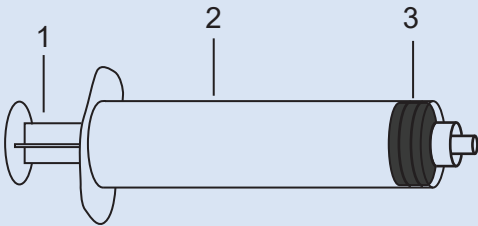

Type	Syringe Size	Ref	Latex or Latex Free
Luer Slip	1ml	436-1-14A	latex
Luer Slip	2ml	436-2-14A	latex
Luer Slip	2.5ml	436-2.5-14A	latex
Luer Slip	3ml	436-3-14A	latex
Luer Slip	5ml	436-5-14A	latex
Luer Slip	10ml	436-10-14A	latex
Luer Slip	20ml	436-20-14A	latex
Luer Slip	30ml	436-30-14A	latex
Luer Slip	50ml	436-50-14A	latex
Luer Slip	60ml	436-60-14A	latex
Luer Lock	1ml	437-1-14A	latex
Luer Lock	2ml	437-2-14A	latex
Luer Lock	2.5ml	437-2.5-14A	latex
Luer Lock	3ml	437-3-14A	latex
Luer Lock	5ml	437-5-14A	latex
Luer Lock	10ml	437-10-14A	latex
Luer Lock	20ml	437-20-14A	latex
Luer Lock	30ml	437-30-14A	latex
Luer Lock	50ml	437-50-14A	latex
Luer Lock	60ml	437-60-14A	latex



Type	Syringe Size	Ref	Latex or Latex Free
Luer Slip	1ml	436-1-14A/LF	Latex Free
Luer Slip	2ml	436-2-14A/LF	Latex Free
Luer Slip	2.5ml	436-2.5-14A/LF	Latex Free
Luer Slip	3ml	436-3-14A/LF	Latex Free
Luer Slip	5ml	436-5-14A/LF	Latex Free
Luer Slip	10ml	436-10-14A/LF	Latex Free
Luer Slip	20ml	436-20-14A/LF	Latex Free
Luer Slip	30ml	436-30-14A/LF	Latex Free
Luer Slip	50ml	436-50-14A/LF	Latex Free
Luer Slip	60ml	436-60-14A/LF	Latex Free
Luer Lock	1ml	437-1-14A/LF	Latex Free
Luer Lock	2ml	437-2-14A/LF	Latex Free
Luer Lock	2.5ml	437-2.5-14A/LF	Latex Free
Luer Lock	3ml	437-3-14A/LF	Latex Free
Luer Lock	5ml	437-5-14A/LF	Latex Free
Luer Lock	10ml	437-10-14A/LF	Latex Free
Luer Lock	20ml	437-20-14A/LF	Latex Free
Luer Lock	30ml	437-30-14A/LF	Latex Free
Luer Lock	50ml	437-50-14A/LF	Latex Free
Luer Lock	60ml	437-60-14A/LF	Latex Free



Caution:

1. Read instruction for use before using the device.
2. This product is sterilized for single use only, discard after single use. Please do not repeat sterilization, do not clean, do not use it once again.
3. Should be used the product immediately after opening the individual package, please destroy when it was used.
4. Do not use it if the inner package has been previously opened or damaged, or package be affected with damp, or the product went moldy.
5. Do not use after expiry date. If over the expiry date, please discard it.
6. Sterile by ETO. Single use only.

Storage:

Store in a cool, dry and dark place. Avoid direct sunlight.

Hypodermic Syringe (without needle)



Secured Medical Direction UK Co., Ltd.

Wayson Commercial Bldg 28 Con-naught Rd West Sheung Wan HK, P.R.C.



Secured Medical Direction UK Co., Ltd.

St. James House 13 Kensington Square W8 5HD London U.K.

www.smd-medical.com