

# Foam Dressing (laminated with PU)



## Instructions For Use:

- Make sure the wound area is clean and dry before using dressing.
- Foam dressing should be 2cm bigger than the wound.
- When the exudates is 2cm close the foam dressing edge, the foam dressing should be changed.
- It can be used together with other dressings.

## Features:

- Material: polyurethane foam.
- Type: laminated with PU.
- Size: 5x5cm, 7.5x7.5cm, 10x10cm, 15x15cm, 20x20cm, 10x20cm.
- High absorbency, it can absorb lots of exudates and reduce maceration to the skin.
- It has water-lock function.
- The semi-permeable polyurethane film can protect the wound against water, bacterial and foreign matters.
- Provide moist environment to promote wound healing.
- Non-adherent to the wound, no pain will occur when remove dressing.
- It is soft, elastic and comfortable to the skin and has good cushioning.
- It is adaptable to different phases of wound, especially to heavy exudated wounds.

I-II degree burn and scald wound. For burn and scald wound, cosmetic operation and shin-donor sites. Ulcer, bedsore, pressure ulcers and cavity wound. Post-operation surgery wounds and injuries.

<b>Size</b>	<b>Ref</b>	<b>Material</b>
5cm x 5cm	180-1-16A	polyurethane foam
7.5cm x 7.5cm	180-2-16A	polyurethane foam
10cm x 10cm	180-3-16A	polyurethane foam
15cm x 15cm	180-4-16A	polyurethane foam
20cm x 20cm	180-5-16A	polyurethane foam
10cm x 20cm	180-6-16A	polyurethane foam



**Caution:**

1. This product is sterilized for single use only, discard after single use.

Please do not repeat sterilization, do not use it once again.

2. Should be used the product immediately after opening the individual package, please destroy when it was used.

3. Do not use it if the inner package has been previously opened or damaged, or package be affected with damp, or the product went moldy.

4. Do not use after expiry date. If over the expiry date, please discard it.

5. Sterile by ETO. Single use only. Latex free.

**Storage:**

Store in a cool, dry and dark place. Avoid direct sunlight.

# Foam Dressing (laminated with PU)



**Secured Medical Direction UK Co., Ltd.**

Wayson Commercial Bldg 28 Con-naught Rd West Sheung Wan HK, P.R.C.

**EC REP** Secured Medical Direction UK Co., Ltd.

St. James House 13 Kensington Square W8 5HD London U.K.

[www.smd-medical.com](http://www.smd-medical.com)