

# Silver Ion Dressing



## Instructions For Use:

- Clean the wound with saline water before using and make the skin around the wound dry.
- Silver ion dressing should not be dipped into saline water before using.
- Silver ion dressing needs to be used together with other dressings such as foam dressing, alginate calcium dressing.
- It is suggested to change the dressing every 3-4 days according to clinical needs.

## Features:

- Material: compound high molecular hygroscopic fiber and silver compound.
- Size: 5x5cm, 7.5x7.5cm, 10x10cm.
- Highly effective and permanent antibacterial ability, and it can kill kinds of pathogenic microorganism. And it is not drug-resistant.
- Can continuously kill bacterial and virus, and this can last for 5-7 days.
- Discharging of silver ion is related to the wound situations. More exudates, silver ion discharging.
- It can protect the wound from bacterial penetration.
- It can promote wound healing and reduce scar formation.
- It will not stick on the wound, and no pain when the dressing is changed. Secondary damage to the wound will not happen.
- It can be used as curing highly exudated infected wound and wounds tending to be infected.  
Can be used for difficult healing chronic wounds (such as diabetic foot wound, venous leg ulcer and so on). Can be filled into deep wound to keep it from being infected. Can be used as anti-infection means for donated and donation wounds.

<b>Size</b>	<b>Ref</b>	<b>Material</b>
5cm x 5cm	206-1-16A	compound high molecular hygroscopic fiber and silver compound
7.5cm x 7.5cm	206-2-16A	compound high molecular hygroscopic fiber and silver compound
10cm x 10cm	206-3-16A	compound high molecular hygroscopic fiber and silver compound



**Caution:**

1. This product is sterilized for single use only, discard after single use.

Please do not repeat sterilization, do not use it once again.

2. Should be used the product immediately after opening the individual package, please destroy when it was used.

3. Do not use it if the inner package has been previously opened or damaged, or package be affected with damp, or the product went moldy.

4. Do not use after expiry date. If over the expiry date, please discard it.

5. Sterile by ETO. Single use only. Latex free.

**Storage:**

Store in a cool, dry and dark place. Avoid direct sunlight.

# Silver Ion Dressing



**Secured Medical Direction UK Co., Ltd.**

Wayson Commercial Bldg 28 Con-naught Rd West Sheung Wan HK, P.R.C.



**EC REP** Secured Medical Direction UK Co., Ltd.

St. James House 13 Kensington Square W8 5HD London U.K.

[www.smd-medical.com](http://www.smd-medical.com)