

Hydrocolloid Plaster




Instructions For Use:


- Take out the hydrocolloid plaster from the package.
- Peel off the back release paper, exposing the adhesive surface.
- The adhesive surface paste on the wound.
- Peel off the PET from the top (for the plaster with PET layer).
- Used for all kinds of skin blisters and corns' plaster. All kinds of acute and chronic small wound. Preventing the friction parts from hurt.

Features:

- Material: adhesive hydrocolloid and high permeability PU film.
- Size: 28x46mm, 37x55mm, 44x69mm, 20x60mm, 25x65mm, 15x50mm, 28x69mm.
- Type: with border or without.
- Supply a moist wound healing environment, and PU is waterproof.
- Protect the wound from infection, comfortable and breathable.
- Different shapes and sizes to provide comprehensive foot protection.

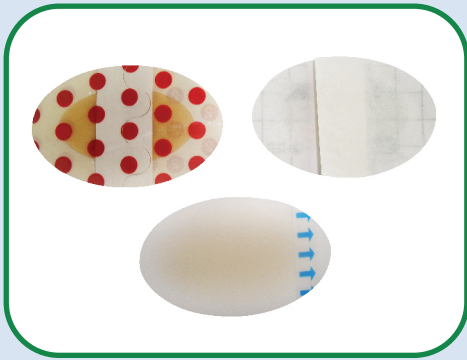

Size	Ref	Type	Material
28mmx46mm	207-1-1-16A	with border	Hydrocolloid, PU
37mmx55mm	207-1-2-16A	with border	Hydrocolloid, PU
44mmx69mm	207-1-3-16A	with border	Hydrocolloid, PU
28mmx46mm	207-2-1-16A		Hydrocolloid, PU
37mmx55mm	207-2-2-16A		Hydrocolloid, PU
44mmx69mm	207-2-3-16A		Hydrocolloid, PU

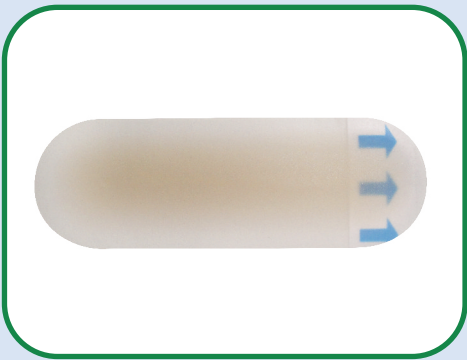

Size	Ref	Type	Material
20mmx60mm	207-1-5-16A	with border	Hydrocolloid, PU
25mmx65mm	207-1-6-16A	with border	Hydrocolloid, PU
15mmx50mm	207-1-7-16A	with border	Hydrocolloid, PU



Size	Ref	Type	Material
28mmx69mm	207-1-4-16A	with border	Hydrocolloid, PU
28mmx69mm	207-2-4-16A		Hydrocolloid, PU

Caution:

1. This product is sterilized for single use only, discard after single use.

Please do not repeat sterilization, do not use it once again.

2. Should be used the product immediately after opening the individual package, please destroy when it was used.

3. Do not use it if the inner package has been previously opened or damaged, or package be affected with damp, or the product went moldy.

4. Do not use after expiry date. If over the expiry date, please discard it.

5. Sterile by ETO. Single use only. Latex free.

Storage:

Store in a cool, dry and dark place. Avoid direct sunlight.

Hydrocolloid Plaster



Secured Medical Direction UK Co., Ltd.

Wayson Commercial Bldg 28 Con-naught Rd West Sheung Wan HK, P.R.C.



EC REP Secured Medical Direction UK Co., Ltd.

St. James House 13 Kensington Square W8 5HD London U.K.

www.smd-medical.com