

Non-woven Self-adhesive Elastic Bandage



Instructions For Use:

- Clean and dry the area to be covered.
- Use a pad or sterile gauze to apply pressure and stop bleeding.
- Cover the wound with the self-adhesive bandage to protect from dust and other pollutants.
- Pull or cut with scissors at the place as required.
- Wrap bandage under tension, covering each turn by a half turn. Make the final turn without tension covering the last turn. Press and hold a few seconds. If bandage is too tight, unwind and reposition until comfortable. Do not apply too tightly.
- This item also can be used for sport protection.

Features:

- Material: non-woven, spandex and viscose.
- Size: 2.5cmx4.5m, 5cmx4.5m, 7.5cmx4.5m, 10cmx4.5m, 15cmx4.5m.
- Soft, breathable, comfortable and high elasticity.
- Different length can be as required.

Size	Unsterile/Ref	Sterile/Ref	Material
2.5cm x 4.5m	73-4.5-1-16B	73-S-4.5-1-16B	non-woven, spandex and viscose
5cm x 4.5m	73-4.5-2-16B	73-S-4.5-2-16B	non-woven, spandex and viscose
7.5cm x 4.5m	73-4.5-3-16B	73-S-4.5-3-16B	non-woven, spandex and viscose
10cm x 4.5m	73-4.5-4-16B	73-S-4.5-4-16B	non-woven, spandex and viscose
15cm x 4.5m	73-4.5-5-16B	73-S-4.5-5-16B	non-woven, spandex and viscose



Caution:

1. This product is for single use only, discard after single use.
2. Do not use it if the package be affected with damp, or the product went moldy.
3. Do not use after expiry date. If over the expiry date, please discard it.
4. Single use only.
5. Sterile or non-sterile, it can be as request.

Storage:

Store in a cool, dry and dark place. Avoid direct sunlight.

Non-woven Self-adhesive Elastic Bandage



Secured Medical Direction UK Co., Ltd.

Wayson Commercial Bldg 28 Con-naught Rd West Sheung Wan HK, P.R.C.



EC REP

Secured Medical Direction UK Co., Ltd.

St. James House 13 Kensington Square W8 5HD London U.K.

www.smd-medical.com